



Proven formula for waterline treatment.

Registered by EPA and FDA and compatible with all dental unit water systems, **Mint-A-Kleen** is a proven liquid formula and protocol that cleans and controls microbial contamination in dental unit waterlines. **Mint-A-Kleen** uses an active ingredient of 0.12% Chlorhexidine Gluconate to directly contact waterline walls and bottles for thorough treatment, cleaning and protection of *waterlines* and *equipment*.

Directions for use* (repeat treatment once a week):

1. Empty water bottle and place back on dental unit. **Air purge waterlines** by activating handpiece hoses and syringes until water is eliminated.
2. **Add 4 oz. of Mint-A-Kleen** (do not dilute) into dental unit's empty water bottle —place back onto dental unit. Activate hoses and syringes until Mint-A-Kleen exits. Allow Mint-A-Kleen to treat waterlines for at least 12 hours.
3. After treatment, remove bottle, empty, then fill with water source. **Flush waterlines with water** for 30 seconds to remove Mint-A-Kleen.

Why choose Mint-A-Kleen?

To learn more about Mint-A-Kleen and the proper cleaning of dental unit waterlines, visit www.MintAKleen.com

It's proven.

Mint-A-Kleen is backed by extensive research and is EPA-registered for use as a dental unit waterline cleaner (EPA 85398-2).

It's easy to use.

There's no need for supplemental products; Mint-A-Kleen is ready to use with no mixing, no fuss, and no mess.

It's economical.

Use just 4 fl. oz. once per week; a single 16 fl. oz. bottle lasts one month per operatory.

It's effective.

Mint blue solution is formulated without harsh chemicals; will not corrode, clog, or leave residue in waterlines, bottles, or equipment.

Mint-A-Kleen is sold in 10-bottle cases. Not available for individual sale. Item #: MAK-OZ.



109 Larrimore Lane | Danville, KY 40422 | USA
P: 866.246.2548 | F: 866.926.8246 | E: info@AnodiaSystems.com
www.AnodiaSystems.com

*See Mint-A-Kleen label insert for full directions for use or visit our Resources page at www.MintAKleen.com. Do not place Mint-A-Kleen bottle directly on dental unit.
©2011 Anodia Systems. All rights reserved. www.AnodiaSystems.com