



MAK-1807 CLOSED WATER BOTTLE SYSTEM

INSTRUCTIONS FOR USE

*Please ensure you have read and followed the installation instructions prior to operation.

PRODUCT DESCRIPTION

An economical way that allows user to choose the water source supply to the dental unit and also permits user to clean dental unit waterlines.

FOR GENERAL DENTAL PROCEDURES

IMPORTANT

Prior to dental unit waterline treatments, remove handpieces, scaler insert and syringe tips.

- 1). After complete installation, flip Toggle switch to ON and continue with general dental procedures.

MAK-1807 CLOSED WATER BOTTLE SYSTEM

INSTRUCTIONS FOR USE

DIRECTIONS FOR WATERLINE CLEANING

STEP 1: AIR PURGE DENTAL UNIT WATERLINES

- Turn dental unit OFF. Empty water bottle and place back on dental unit.
- Turn dental unit ON.
- Air purge waterlines by activating handpiece hoses and air/water syringes until water is eliminated.

STEP 2: ADD MINT-A-KLEEN®

- Turn Dental Unit OFF. Remove water bottle, pour 4 ounces of Mint-A-Kleen into dental unit's empty water bottle, swish Mint-A-Kleen in bottle and then place water bottle on dental unit.
- Turn dental unit ON. Activate handpiece hoses and air/water syringes until Mint-A-Kleen exits.
- Turn dental unit OFF. Allow Mint-A-Kleen to treat waterlines and bottle for at least 12 hours before proceeding to Step 3.

STEP 3: FLUSH DENTAL UNIT WATERLINES

- Remove water bottle, dispose of any excess Mint-A-Kleen and fill bottle with water source.
- Turn dental unit ON. Flush waterlines with water for 30 seconds to remove any excess Mint-A-Kleen.
- Continue with dental procedures.

* Note – For optimal cleaning, air purge and clean all instruments and tubing that contain water using Mint-A-Kleen and its protocol. Additionally, clean any areas that cannot be reached with normal protocol which are called 'Dead-Ends'. These dead-ends can be opened and cleaned with a Drain Tube.

* Activate all the water outlets including the quick-disconnect water outlet for the cavitron until air comes out. Use the Drain Tube to open quick-disconnect water outlet for the cavitron. Connect Drain Tube to the saliva ejector first and then to the quick-disconnect water outlet. After air purging is completed, disconnect the Drain Tube.